# pH-Fix 4,5 – 10,0 PT

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# pH-Fix 4.5 – 10.0 PT

## Test strips for measuring the pH values in the range of pH 4.5 – 10.0

 Please refer to instructions for use

**REF** 92121

**Pack content:**

100 test strips

1 reference color scale pH 4.5 – 10.0 PT

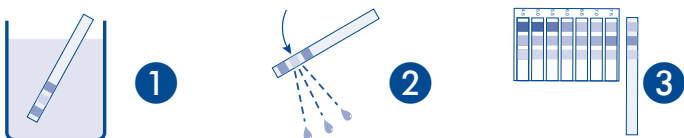
**Application:**

The pH-Fix test strips pH 4.5 – 10.0 PT are semi quantitative indicator test strips for many applications.

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**Instruction:**

1. Dip the test strip with all its fields into the sample solution for 1 sec.
2. Shake off excess liquid.
3. Compare the colors with the color chart and read off the pH value which matches closest.

**Remark:**

In weakly buffered solutions, keep the test strip immersed for a longer period of time until color consistency of all test fields.

**Storage:**

Avoid exposing the strips to sunlight and/or moisture. Keep container cool and dry (store between 4 – 30 °C).

**Disposal:**

Used test strips may be disposed of as domestic waste.

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[www.mn-net.com](http://www.mn-net.com)

## pH-Fix 4.5 – 10.0 PT

Test strips for measuring pH values in gastric juice in the range of pH 4.5 – 10.0

  IVD for qualified professional only  Please refer to instruction for use

 92121

### Pack Content:

100 Test strips  
1 Color chart pH 4.5 – 10.0 PT

### Application:

The pH-Fix 4.5 – 10.0 PT test strips are semi-quantitative indicator strips for measuring the pH value of gastric juice in the range of 4.5 – 10.0 with a gradation of 0.5. The test strips can be read visually by comparison with the color chart.

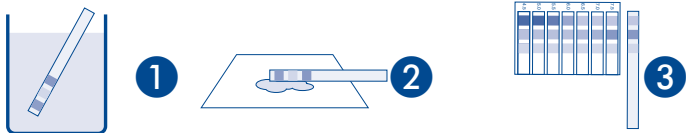
For quality control use commercially available pH buffer solution.

### Caution:

The pH-Fix 4.5 – 10.0 PT test strips for measuring the pH-value in gastric juice are only intended to be used by qualified professionals! For quality control use commercially available pH buffer solution.

### Instructions for measuring the pH value of gastric juice:

1. Collect a sample in a clean vessel. Remove test strip and dip it with all its test fields into the sample until the color stops changing.
2. Dab the strip along the edge on an absorbent cloth to remove excess liquid.
3. Compare the test fields with the color scale (accuracy of reading  $\pm 0.2$  pH).

### Performance study:

A comparative study (N = 390) shows an average deviation of the test strip compared to the reference method (pH electrode) of 0.09 pH units.

### Storage conditions:

Avoid exposing the strips to sunlight and/or moisture. Keep container cool and dry (store between 4 – 30 °C). The test strips are stable, when stored properly up to the date of expiry indicated.

### Disposal:

Used test strips may contain potentially infectious material of human origin. Dispose all used test strips in accordance to the local laws and regulations.

### Note:

Do not touch the test fields.

Test strips are for single use only.

The use of the system is limited to samples that are not strongly colored.

Sample volume must be sufficient to properly wet all test pads according to the instructions for use.

Only for use outside of the body.

### Test principles:

The pH value of the sample generates a color change of the pH indicator dye that is chemically bound to the paper of the test fields.

### Reporting:

Please note that any serious incident that has occurred in relation to the product shall be reported immediately to the manufacturer and the competent authority of the European member state in which the incident occurred. European vigilance contact points:

[http://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](http://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

### Literature:

A. M. Bourgault, J. Heath, V. Hooper, M. L. Sole, E. G. NeSmith, *Critical Care Nurse*, 2015, 35.

 Statement of Conformity (Product corresponds to the In-Vitro Diagnostic Directive 98/79/EC of the European Union)

 In-vitro Diagnostic

 Please read instructions for use!

 Permitted storage temperature range

 Use by

 Batch identification

 Item number

 Manufacturer

 Do not reuse

### Technical Service

If you have any questions after reading the manual or if you need technical assistance, please contact

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MACHEREY-NAGEL GmbH & Co. KG